

# ManaEZ BOOT AIR SHORT

Plastic Camboot with Pneumatic Pump

## INSTRUCTIONS FOR USE

### Features

- **Rocker bottom to simulate natural gait**
- **Pneumatic Pump and Short Design**
- **Large air bladder creates a larger surface for contact pressure to help control swelling and ensure a snug fit**



### To put on the ManaEZ Boot Air Short:



1. Open liner by unfastening straps and removing front shell. Place lower leg and foot in brace. Wrap liner.


Optional: Foam pads may be placed inside liner or between liner and shell for a better fit or greater comfort.



3. Fasten foot, ankle straps. Tighten straps until snug and comfortable.




2. If necessary, using hook attachments, fix the toe cover to the liner.

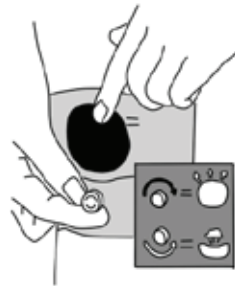

### Using the AirValve:

4. For inflation, ensure the round valve is closed by turning the valve clockwise until it is secure. Pump the air bulb until you feel pressure within the boot. For deflation, turn the round valve counter-clockwise and wait 15 seconds until the air is released.

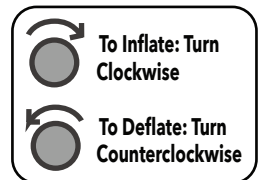

Product Name	Men's Shoe Size	Women's Shoe Size	Part Number
ManaEZ Boot Air Short Extra Small	Up to 5.5	Up to 4	SEZBA01XS
ManaEZ Boot Air Short Small	6 - 8	4 - 7	SEZBA01S
ManaEZ Boot Air Short Medium	8 - 11	7 - 10	SEZBA01M
ManaEZ Boot Air Short Large	11 - 13	10 - 12	SEZBA01L
ManaEZ Boot Air Short XLarge	13 +	12 +	SEZBA01XL

**Warning:** This device is intended for use only under the advice of a medical professional.

If you experience any pain, swelling, sensation changes, or any unusual while using this product, consult your medical professional immediately.



**MANAMED™**

ManaMed Inc.  
5240 W Charleston Blvd.  
Las Vegas NV 89146  
United States  
Ph: 888-508-0712  
www.ManaMed.com

©2020 ManaMed Inc. All rights reserved.  
ManaMed, the ManaMed logo,  
EZ Boot Air Short logo, are trademarks of  
ManaMed Inc. Registered in the U.S.